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LYMED® wear and care instructions

Read the instructions carefully before starting pressure treatment. Keep the instructions safe for the entire duration of the treatment. Washing before first use is recommended.

Lymed Oy is a manufacturer of high class techno-medical garments with over 20-year experience. Carefully chosen materials are used in manufacturing enabling us to design pressure garments that are comfortable to wear, look good and give the best possible result in pressure treatment.

Your product has left our factory inspected and well-packaged. If you have any questions concerning your product, please contact your own Lymed® therapist or measurer.

Wear

The product is safe to use round-the-clock, leaving time for personal hygiene and skin care. The wear time of the product is determined by the person responsible for the treatment. If unusual swelling, circulatory disorders, breakdown of the skin or other symptoms occurs, please contact the persons responsible for your care.

Familiarize yourself with the instructions and learn how to put your garment on. If the product feels uncomfortable or the wrong size, it might be put on incorrectly. If the fit is perceived to be changing as time goes by, body measurements should be checked and the product re-fitted. The feeling of the product will change after first wash when the composition of the fabric adjusts.

Product's service life is shortened by e.g. using the product while swimming, using fabric softeners when washing or long term exposure to sunlight. Protect the product with another piece of clothing when it is exposed to unusual stress. Lymed Oy will only compensate the repair expenses of a product or completely replace the products which are chosen per recommendations and used per instructions. Lymed Oy withholds the right to either repair or replace the defective product at their own discretion.

Wash and care

Product must be washed daily because a well-maintained product will keep its qualities longer. With correct and careful use you can influence the outcome of the treatment. Colors may fade or release pigment.

- Wash daily (40°C). For example sweat, sebum, lotions and deodorant will destroy the elastane fiber thus hurting the material.
- The use of fabric softener and washing detergent containing fabric softener is not allowed.
- Use a washing bag and wash with similar colors.
- Close all possible zippers/velcros/fasteners when washing.
- Wash right after swimming.
- Only line dry, dry flat or tumble dry (at low temperatures) are allowed.
- Products must be clean and dry when sent for repairs.

Do not remove the label attached to the garment. It contains vital information for possible product traceability and repairs. Thank you!

Videos on how to put on garments at www.lymed.fi

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